

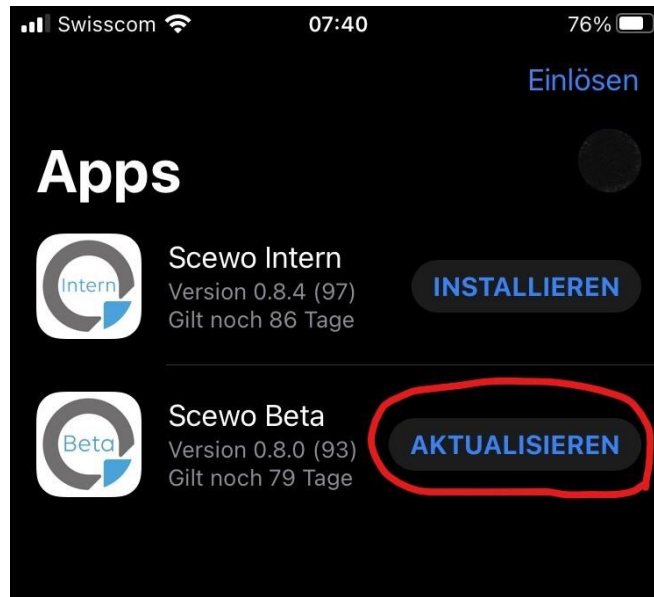
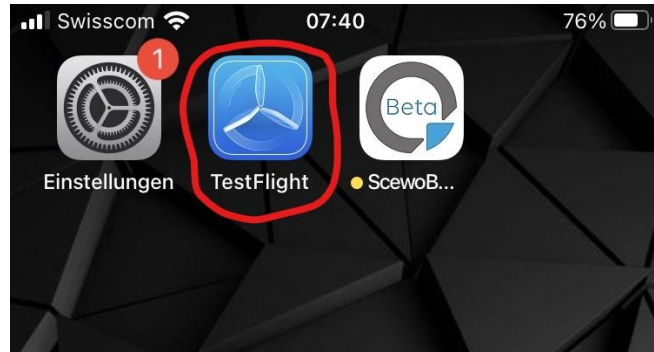


Instruction App Update (Beta only)


This instruction is only for users with the beta version of the BRO-App

<p>1. Check that the Wi-Fi is turned on and connected to the one at home, not to the one from your BRO. To update your app, you need internet access.</p>	 <p>The screenshot shows the 'Einstellungen' (Settings) app on an iPhone. The 'WLAN' option is highlighted with a red circle. The status bar at the top shows 'Swisscom' as the carrier, a Wi-Fi icon, the time '07:39', and '76%' battery. Other settings visible include 'Flugmodus' (Airplane Mode), 'Bluetooth', 'Mobiles Netz' (Cellular), and 'Persönlicher Hotspot'.</p>
<p>2. Make sure that you are using the latest operating system (iOS 14.2). You can check this in the settings under "General" at "Software update".</p>	 <p>The first screenshot shows the 'Allgemein' (General) settings page with 'Softwareupdate' (Software Update) highlighted by a red circle. The second screenshot shows the 'Softwareupdate' page, which displays 'iOS 14.2' and the message 'Deine Software ist auf dem neuesten Stand.' (Your software is up to date).</p>


3. Go back to the Home screen and open the TestFlight app. Click on the "Update" field in Scewo Beta, the app will now be updated. If the app already says "Open", it is up to date.

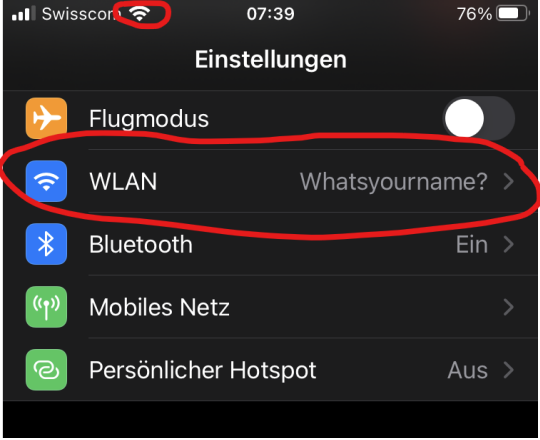

4. Now click on the Scewo Beta app icon and scroll down to "automatic updates"..

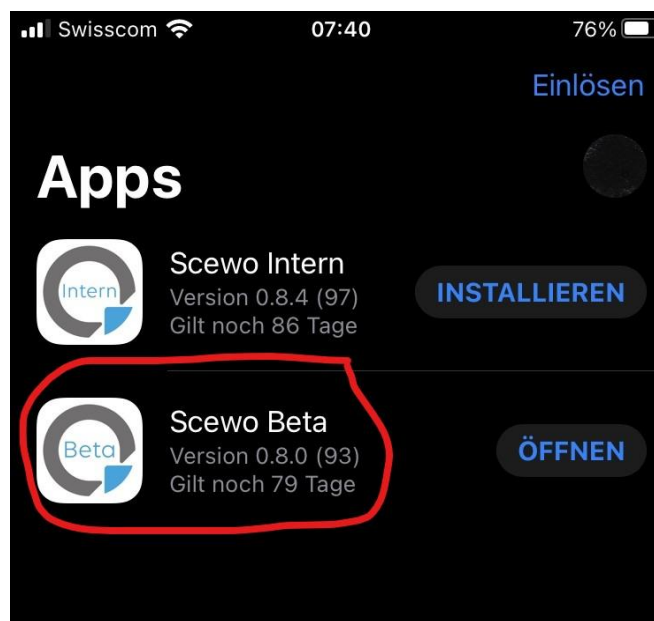

- a. Switch on the automatic update here. The app will now always be updated when you are connected to the home Wi-Fi.

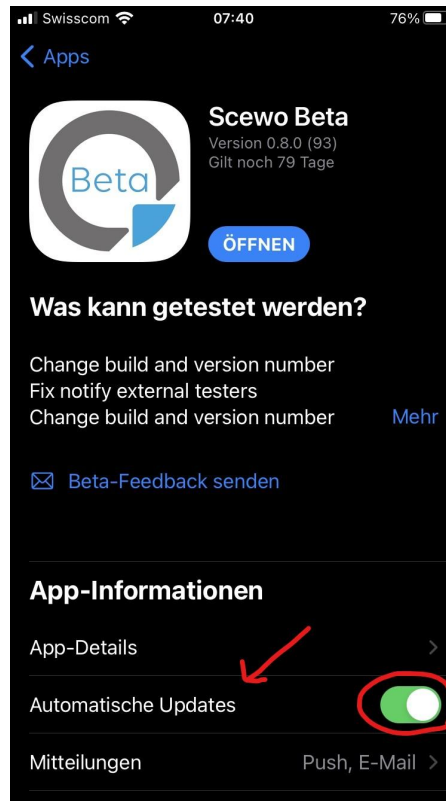

5. Open the Scewo Beta app via the "Open" button and check out the new features.

