


# BRO as luggage.

Find out if you can stow away BRO in your car.

BRO was designed to be easily transported in a wide variety of vehicles. The backrest can be folded down or alternatively removed. This allows it to fit into even smaller storage spaces. With this guide, you can easily check whether BRO can be stowed away in your car. Follow the three steps below.

Are you still unsure whether BRO will fit in your car after measuring?  
We're happy to help you, just contact us at [sales@scewo.ch](mailto:sales@scewo.ch).

- 1 Measure the depth (D) of the boot in your car.
- 2 Measure the height (H) of the boot in your car.<sup>1</sup>
- 3 Search for the boot dimensions you have determined in the table and find out if BRO fits.

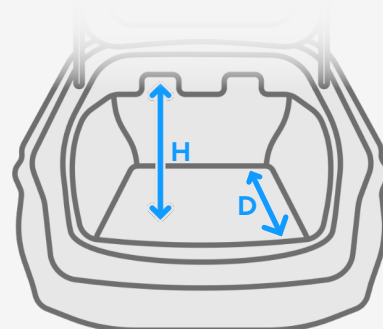

<sup>1</sup> Make sure you measure the smallest height (loading hatch).

		Depth (D)			
		Up to 107.5 cm	107.5 – 113 cm	113 – 119.5 cm	From 119.5 cm
Height (H)	Up to 67 cm	✗	✗	✗	✗
	67 – 72.5 cm	✗	✓	✓	✓
	72.5 – 73.5 cm	✗	✓	✓	✓
	73.5 – 79 cm	✗	✓	✓	✓
	From 79 cm	✗	✓	✓	✓

- ✗ Unfortunately, BRO does not fit in your car.
- ✓ BRO fits in your car with the backrest removed.
- ✓ BRO fits in your car with the backrest folded down.