

Are my stairs suitable?


Find out if BRO can drive up/down your stairs.



BRO's unique tracks can climb up and down a wide range of stairs up to 36°.¹ With this guide you can easily check if BRO is suitable for your stairs. Follow the three steps below.

Are you still unsure after measuring whether BRO can climb your stairs? We're happy to help you, just contact us at sales@scewo.ch.

- 1 Measure the depth (D) of the step and write the number down.²
- 2 Measure the height (H) of the step and write the number down.³
- 3 Find the dimensions of the step in the table and see if BRO can drive up/down these stairs.

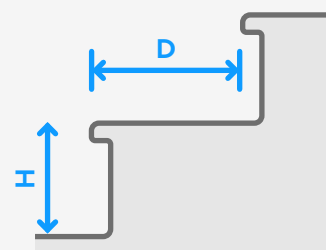

		Depth (D)															
		20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35
Height (H)	14	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
	15	✗	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
	16	✗	✗	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
	17	✗	✗	✗	✗	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
	18	✗	✗	✗	✗	✗	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
	19	✗	✗	✗	✗	✗	✗	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
	20	✗	✗	✗	✗	✗	✗	✗	✗	✓	✓	✓	✓	✓	✓	✓	✓
	21	✗	✗	✗	✗	✗	✗	✗	✗	✗	✗	✗	✗	✗	✗	✗	✗

¹ Spiral staircases are not accessible with BRO.
 ² The landing depth after the flight of stairs must be at least 120 cm.
 ³ Single steps are accessible up to a maximum height of 20 cm.