

# BRO Updates.

How to download the latest software to your BRO.


Thanks to regular software updates, BRO is becoming more intelligent and constantly learning new functions. Here you will find an overview of the update process, so that you and your BRO can benefit from the latest improvements.

Are you still unsure how to update the BRO software despite the guide? We're happy to help you. You can find the contact details of Scewo and our partners on our website.

1

Make sure your smartphone is connected to the internet and not to your BRO.


The app will automatically check if an update is available. To do this, go to the settings and to the «BRO Software» section.

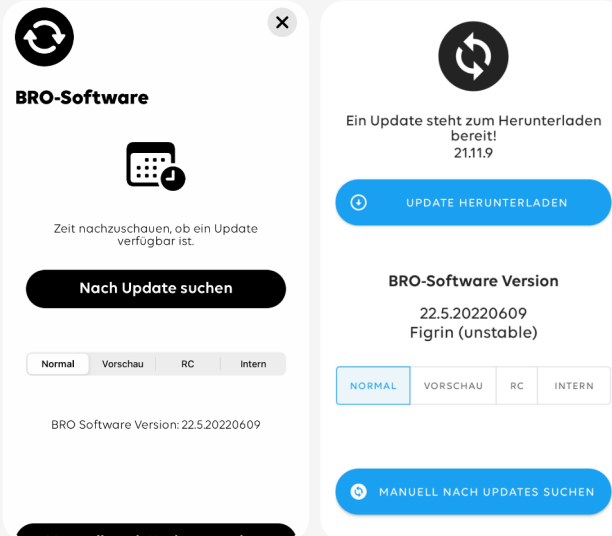

2

Click on «Check for update» and click on «Download update» if an update is available.



The update will now download, which takes about a minute.

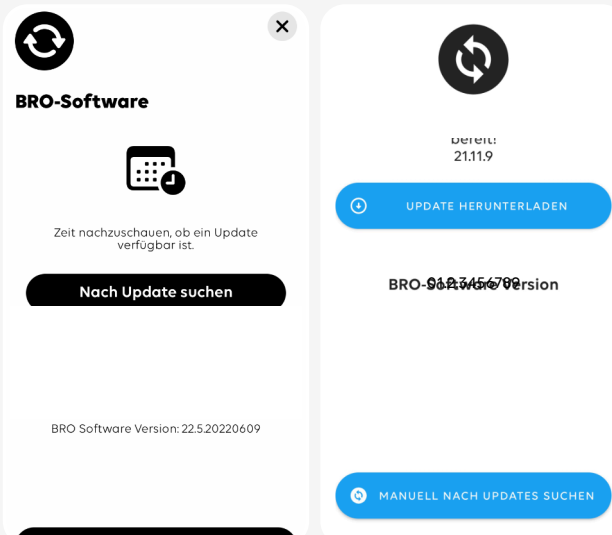

3



In order for the new software to be installed on BRO, the following requirements must be met:

1. the app must be connected to BRO.
2. BRO must be in park mode.
3. the battery of BRO must be at least 20% charged.

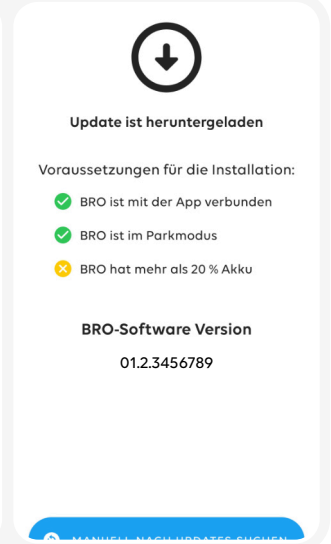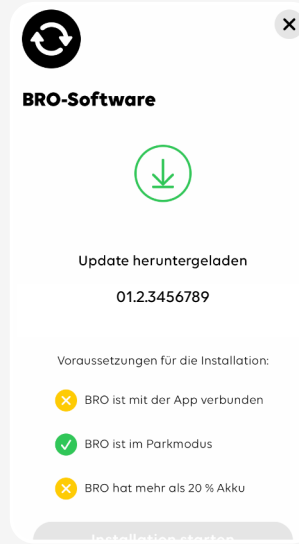



4

When all requirements are met, click on the «Start installation» button and your BRO will be updated.

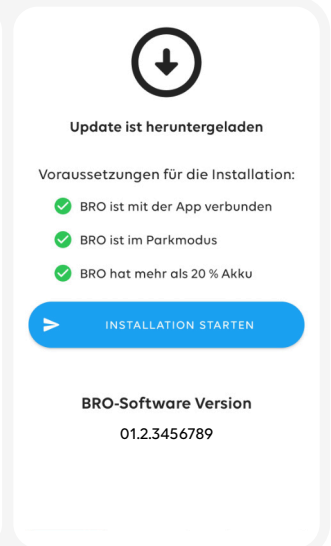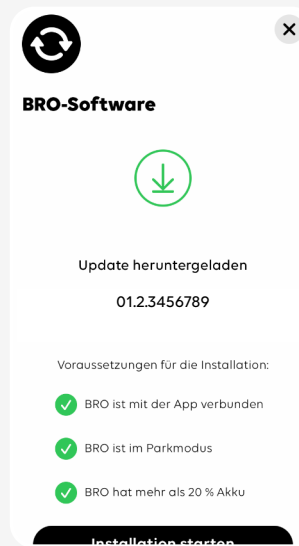


5

The update is first sent to your BRO and then installed directly.

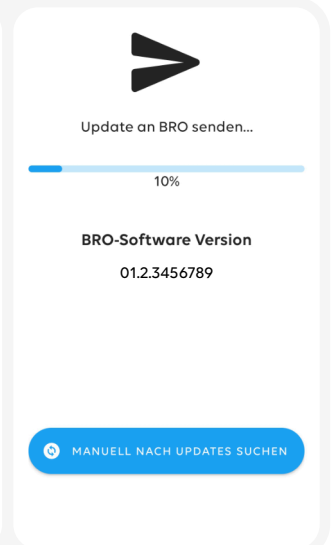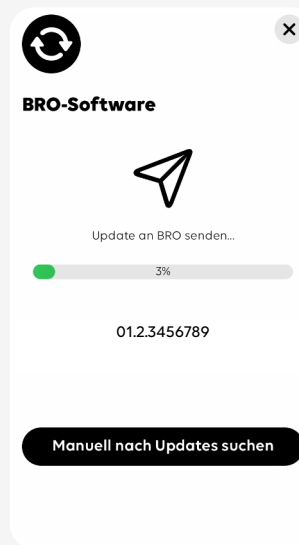

6

Your BRO restarts automatically.  
Reconnect it to the app as soon as the control panel lights up green.

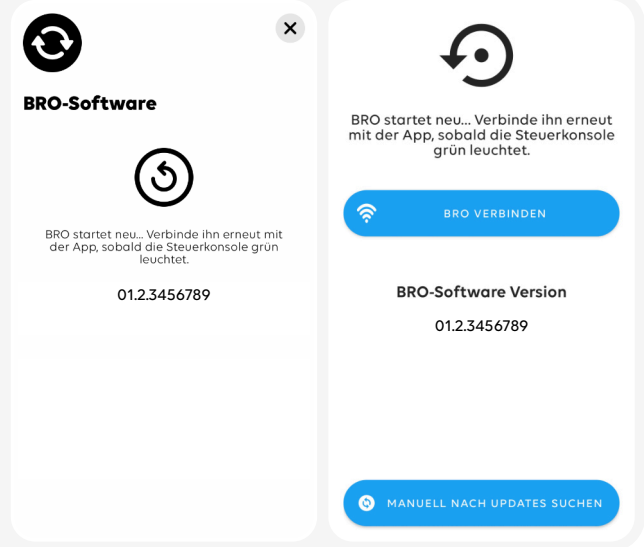

7

Your BRO is now up to date and ready for the next adventure.

In the settings under «BRO Software» you can see which software version is installed on your BRO.

